



Aqua Eve Female Urinal Directions

1. Place over peri area with flat side down. Ensure that the entire surface is in contact with your body to limit drips
2. Hold firmly in place during use
3. Swipe up when done to capture as much moisture as possible
4. Dry off the surfaces, then firmly press the cap down over the ridge on the soft contact ring. Please dry thoroughly to ensure a good fit
5. Please allow for some practice, this unique design has been tested to work with the female anatomy but many of us are first timers!
6. Clean between uses with soapy water or disinfectant wipes. Once a week a bottle brush can be used with either vinegar and water, toilet bowl cleaner or any household cleaner spray for a more rigorous clean

