











B5040 Toilet Safety Rails

Included in box:

A  Mounting Bracket	B  Curved Bar #1	C  Curved Bar #2	D  x 2 Armrest
E  x 2 Leg	F  x 2 Bolt	G  x 2 Curved Washer	H  x 2 Wingnut

USER WEIGHT LIMIT: 300lbs.

ATTENTION:


- Please verify that all components are included as shown above, before proceeding with assembly.
- Make sure to read and follow these instructions and warnings carefully. Failure to do so may result in an injury.
- Persons with severe disabilities or persons who are unable to walk without an aide should not attempt to use this product without assistance. Consult a healthcare professional for advice and instructions specific to your needs before use.
- Check after installation to make sure the rail is secure.
- Inspect all parts carefully for cracks or breaks and ensure all push buttons are properly engaged in position before use.
- Evenly distribute weight between armrests during use.
- Periodically check and maintain the condition of the toilet seat bolts to ensure that they are not damaged. If there is a concern about the strength and condition of the current toilet seat's hardware, replace it with an appropriate new set.

March 2023, ESSENTIAL MEDICAL SUPPLY, INC.®



Assembly Instructions

- 1 Remove the existing seat and lid from the toilet; clean around the toilet's mounting holes. Make sure to save the existing toilet seat's hardware as they will be needed in Step 3.



- 2 Take the Mounting Bracket (A) and place its flat side down onto the toilet bowl; align the two slotted holes in the Mounting Bracket (A) with the toilet's seat mounting holes. The two sets of 3 holes in the tubular portion of the Mounting Bracket (A) should be facing forwards.

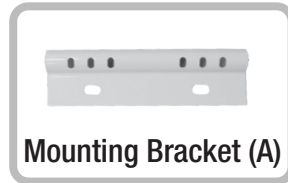

- 3 Install the toilet seat over the Mounting Bracket (A) by using the existing toilet seat's hardware. Ensure the hardware is securely fastened.

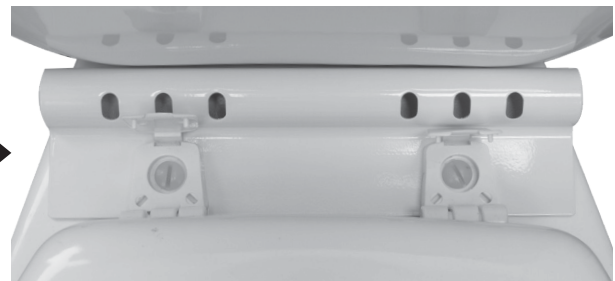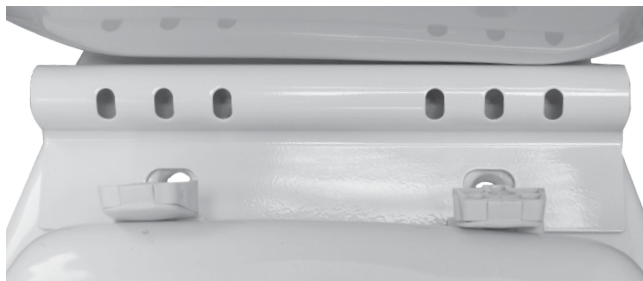



- 4 Take Curved Bar #1 (B) and insert the tubing with the push button into the Mounting Bracket (A) until the push button protrudes from one of the three slotted holes. Make sure the push button is fully protruding from one of the slotted holes and is facing forwards.

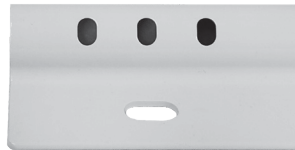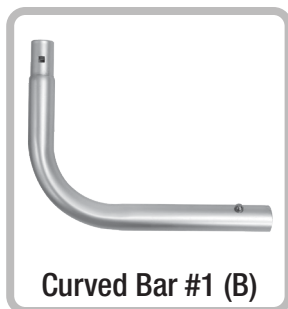

Repeat with Curved Bar #2 (C) and insert it into the other side of the Mounting Bracket (A).





- 5** Take one of the Armrests (D) and one of the Legs (E). Press in the push button found towards the bottom of the Armrest (D) and slide the Leg (E) over the Armrest (D) tubing until the push button protrudes from adjustment hole number 1 [the height of the Leg (E) can be adjusted at a later time].

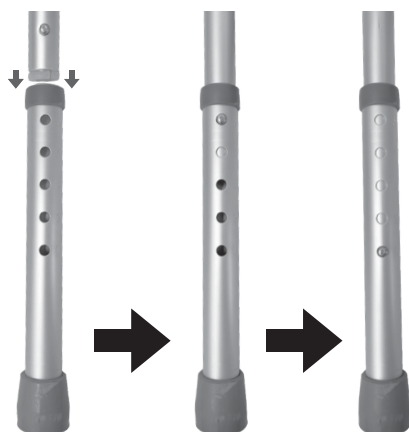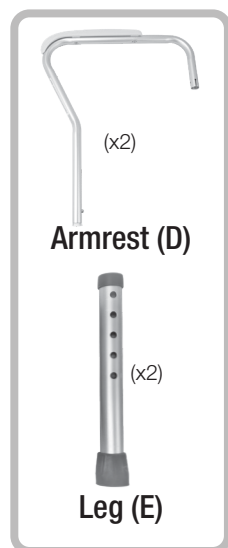

Repeat assembly with the other Armrest (D) and Leg (E).

- 6** Take one of the Armrests (D) [with the Leg (E) installed] and place the open end over one of the Curved Bars (B or C). Make sure to align the square-shaped holes in each part.

Insert one of the Bolts (F) into the square-shaped hole, so that the Bolt's (F) head is facing forwards. Take the Curved Washer (G) and install it over the end of the Bolt (F) so that its curvature aligns with the tubing. Take a Wingnut (H) and thread it onto the end of the Bolt (F).

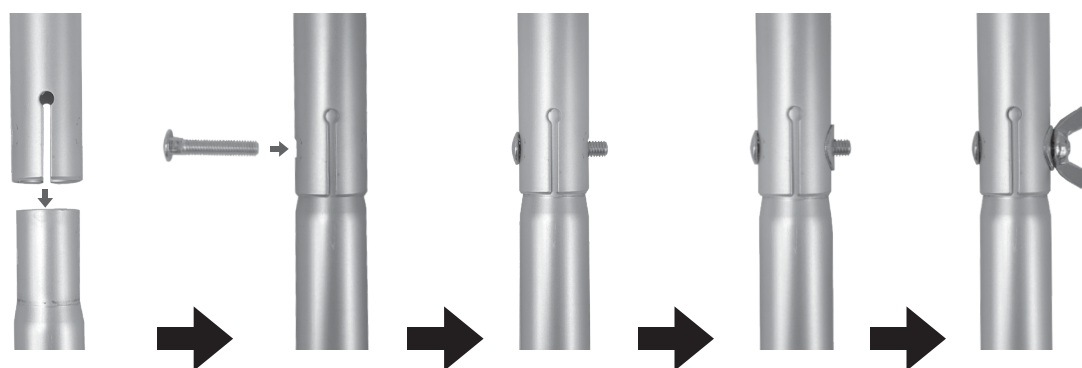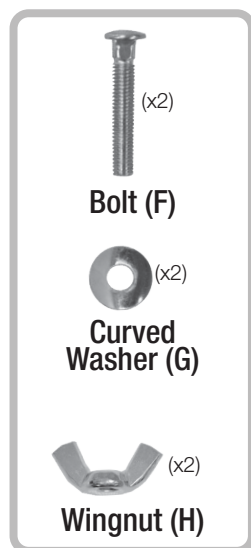

Repeat installation onto the other Curved Bar (B or C) with the other assembled Armrest (D) and Leg (E). Ensure the Wingnuts (H) are tight.

- 7** If needed, adjust the Legs (E) so that the Armrests (D) are level and at a comfortable level.

Also, adjust the Armrests (D) inwards or outwards to desired width by sliding the Curved Bar (B/C) + Armrest (D) assemblies to the slotted holes in the Bracket (A).

You have now completed the assembly of your Toilet Safety Rails.



Essential Medical Supply, Inc. Limited Warranty

Essential Medical Supply, Inc. ("Essential") warrants its products free from defects in materials and workmanship ("Warranty") for a period of one (1) year from the date of purchase ("Warranty Period"). The Warranty extends only to the original purchaser of Essential products. If within the Warranty Period there is a breach of Warranty as to an Essential product, such product shall be repaired or replaced at Essential's sole discretion. This Warranty does not include any labor or shipping costs incurred in replacement, part installation, or repair of any Essential product. **ESSENTIAL'S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO REPAIR AND/OR REPLACEMENT OF THE DEFECTIVE ESSENTIAL PRODUCT.**

For Warranty service, please contact the dealer from whom you purchased your Essential product. In the event you do not receive satisfactory Warranty service, please write Essential directly at:

Essential Medical Supply, Inc.
Customer Service Department
6420 Hazeltine National Drive
Orlando, FL 32822-5121

Please provide the dealer's name, address, and date of purchase, and indicate the nature of the problem and the Essential product serial number.

Upon receipt of written notification, Essential will issue a return authorization. The defective unit or parts must be returned to Essential for Warranty inspection using the serial number as identification within thirty (30) days of the date of return authorization. **DO NOT** return products to Essential without Essential's prior written consent. Cash on delivery shipments will be refused and shipping charges must be prepaid.

Limitations and Exclusions: The Warranty shall not apply to products if 1) the Essential serial number has been removed or defaced; 2) the product has been damaged by negligence, accident, improper operation, maintenance or storage; and/or 3) the product has been modified without Essential's express written consent or repaired by an unauthorized third-party ("Warranty Disqualifying Events"). The Warranty Disqualifying Events include but are not limited to the addition of unauthorized parts or attachments or damage caused by circumstances beyond Essential's control, the occurrence of which shall be determined exclusively by Essential. The Warranty also shall not apply to problems or defects arising from normal wear and tear on Essential products and parts, including but not limited to rubber tips and hand grips, and failure to adhere to user instructions and safety precautions.

THE WARRANTY STATED HEREIN IS THE SOLE AND EXCLUSIVE WARRANTY OFFERED BY ESSENTIAL. THERE ARE NO OTHER WARRANTIES RESPECTING ESSENTIAL'S PRODUCTS, EITHER EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO ANY WARRANTY OF DESIGN, MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE, EVEN IF ESSENTIAL HAS BEEN INFORMED OF SUCH PURPOSE. NO AGENT OF ESSENTIAL IS AUTHORIZED TO ALTER OR EXCEED ESSENTIAL'S WARRANTY OBLIGATIONS AS SET FORTH HEREIN. ESSENTIAL SHALL NOT BE RESPONSIBLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL, OR CONSEQUENTIAL DAMAGES ARISING OUT OF USE OF ESSENTIAL PRODUCTS.

Product Serial No. _____