

ENDURANCE®

BUILT TO LAST!

Quad Cane



ITEM #

W1301 Small Base

W1302 Large Base

Made in China

2012 ESSENTIAL
MEDICAL SUPPLY, INC.®



Manufactured for
ESSENTIAL®
MEDICAL SUPPLY, INC.
Orlando, FL 32822

— SAVE THESE INSTRUCTIONS —

Congratulations on the purchase of your new Endurance® Quad Cane. With proper care, this quality product will give you years of service.

Do not attempt to adjust or operate your Endurance® Quad Cane without carefully reading all instructions in this manual. If, after reading this manual, you have any questions, please call your health professional or place of purchase, otherwise injury or damage may occur.


To insure correct use of cane, the longest portion of cane base (C) must be directly parallel to your foot. Handle can be reversed to provide positioning on whichever side you need to use it on.

TO CHANGE TO LEFT OR RIGHT HANDED USE:

1. Loosen Knob **(A)**
2. Depress the Brass Button **(B)** to clear the hole in the shaft of the upper half of the cane and turn the shaft until the handle is in the Right or Left Hand use position.
3. Make sure that the Brass Button **(B)** snaps back into the hole in the shaft.
4. Tighten Knob **(A)**
5. After making any adjustment, always check to confirm that the Knob is fully tightened before using the cane.

TO ADJUST HEIGHT:

1. Loosen Knob **(A)**
2. Depress the Brass Button **(B)** to clear the hole in the shaft of the upper half of the cane and adjust shaft Up or Down to desired height.
3. Make sure that the Brass Button **(B)** snaps back into the hole in the shaft.
4. Tighten Knob **(A)**
5. After making any adjustment, always check to confirm that the Knob is fully tightened before using the cane.

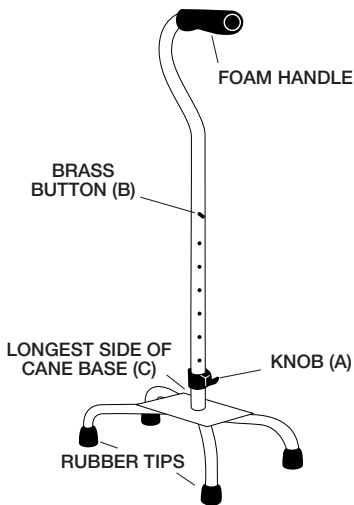

! CAUTION: !

Endurance® Quad Canes have a user weight limit of 300 lbs.

Before using the cane, make sure that:

- The **Brass Button** is fully protruding through the hole in the shaft.
- The **Knob** is fully tightened.
- The **Rubber Tips** are not worn, ripped or missing.
- Always take short steps. Over striding can cause loss of balance.
- Do not attempt to negotiate stairs with any type of cane without the approval of your physician or therapist.

ESSENTIAL MEDICAL SUPPLY, INC. LIMITED WARRANTY

Essential Medical Supply, Inc. ("Essential") warrants its products free from defects in materials and workmanship ("Warranty") for a period of three (3) years from the date of purchase ("Warranty Period"). The Warranty extends only to the original purchaser of Essential products. If within the Warranty Period there is a breach of Warranty as to an Essential producer, such product shall be repaired or replaced at Essential's sole discretion. This Warranty does not include any labor or shipping costs incurred in replacement, part installation, or repair of any Essential product. ESSENTIAL'S SOLE OBLIGATION AND YOUR EXCLUSIVE REMEDY UNDER THIS WARRANTY SHALL BE LIMITED TO REPAIR AND/OR REPLACEMENT OF THE DEFECTIVE ESSENTIAL PRODUCT.

For Warranty service, please contact the dealer from whom you purchased your Essential product. In the event you do not receive satisfactory Warranty service, please write Essential directly at:

Essential Medical Supply, Inc.
Customer Service Department
6420 Hazeltine National Drive
Orlando, FL 32822-5121

Please provide the dealer's name, address, and date of purchase, and indicate the nature of the problem and the Essential product serial number.

Upon receipt of written notification, Essential will issue a return authorization. The defective unit or parts must be returned to Essential for Warranty inspection using the serial number as identification within thirty (30) days of the date of return authorization. DO NOT return products to Essential without Essential's prior written consent. Cash on delivery shipments will be refused and shipping charges must be prepaid.

Limitations and Exclusions: The Warranty shall not apply to products if 1) the Essential serial number has been removed or defaced; 2) the product has been damaged by negligence, accident, improper operation, maintenance or storage; and/or 3) the product has been modified without Essential's express written consent or repaired by an unauthorized third-party ("Warranty Disqualifying Events") include but are not limited to the addition of unauthorized parts or attachments or damage caused by circumstances beyond Essential's control, the occurrence of which shall be determined exclusively by Essential. The Warranty also shall not apply to problems or defects arising from normal wear and tear on Essential products and parts, including but not limited to cane feet, handles, straps and failure to adhere to user instructions and safety precautions.

THE WARRANTY STATED HEREIN IS THE SOLE AND EXCLUSIVE WARRANTY OFFERED BY ESSENTIAL. THERE ARE NO OTHER WARRANTIES RESPECTING ESSENTIAL'S PRODUCTS, EITHER EXPRESS OR IMPLIED, INCLUDING BUT NOT LIMITED TO ANY WARRANTY OF DESIGN, MERCHANTABILITY, OR FITNESS FOR A PARTICULAR PURPOSE, EVEN IF ESSENTIAL HAS BEEN INFORMED OF SUCH PURPOSE. NO AGENT OF ESSENTIAL IS AUTHORIZED TO ALTER OR EXCEED ESSENTIAL'S WARRANTY OBLIGATIONS AS SET FORTH HEREIN. ESSENTIAL SHALL NOT BE RESPONSIBLE FOR ANY INDIRECT, INCIDENTAL, SPECIAL, OR CONSEQUENTIAL DAMAGES ARISING OUT OF USE OF ESSENTIAL PRODUCTS.

Product Serial No. _____

NOTE: Warranty voids if Serial No. label is removed or destroyed.