



# Etac Clean

Shower commode chairs

 **etac**<sup>®</sup>

# Etac Clean

Etac Clean is secure, safe and simple for both users and carers. With practical functionality and design, every detail of the chair has been developed to give the user a positive and safe bathroom experience.

## Easy to fit and manoeuvre

Etac Clean is extremely easy to operate, thanks to the width of the push handle. The clever design can be seen throughout the entire chair. The upright handle ensures that Clean is a perfect fit, even over wall-mounted toilets, and the chair is open at the back to facilitate access.

## Comfortable and practical foot support

The foot support is unique and can be pushed under the seat when not in use.

The soft curved surface provides good comfort for the feet, which are also protected by the foot support's side protection.

All other variants of Clean also have these features, but with additional functionality to meet other needs.



### Etac Clean shower commode chair

	White	Lagoon green
44 cm (17¼"), 4 lockable castors	80229269	80229209
49 cm (19¼"), 4 lockable castors	80229270	80229210
49 cm (19¼"), 2 lockable castors	80229273	80229213
49 cm (19¼"), 4 lockable wheels, backrest upholstery, panholder	80229286	-
49 cm (19¼"), 2 lockable castors, soft back	80229305	-
55 cm (21¾"), 4 lockable wheels, backrest upholstery, panholder	80229287	-
55 cm (21¾"), 4 lockable castors	80229271	80229211
55 cm (21¾"), 2 lockable castors	80229274	80229214
55 cm (21¾"), 2 lockable castors, soft back	80229306	-
60 cm (23½"), 2 lockable castors incl. heightening spacers	80229275	80229215

### Etac Clean shower commode chair - double box

	White	Lagoon green
49 cm (19¼"), 4 lockable castors, double box	-	80229220
49 cm (19¼"), 2 lockable castors, double box	80229283	80229223
55 cm (21¾"), 4 lockable castors, double box	80229281	80229221
55 cm (21¾"), 2 lockable castors, double box	80229284	80229224

#### Weight

44 cm: 13.5 kg  
49 cm: 14.0 kg  
55 cm: 14.5 kg  
60 cm: 15 kg

#### Maximum user weight

130 kg

#### Facts



#### Material

Chassis: Powder-painted steel tubing  
Seat/back support: Polypropylene  
Arm supports: Polypropylene  
Foot support: Polypropylene  
Caster brackets: Polyamide  
Castors, lockable, ball-bearing: Polyamide/TPE

Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.  
The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.

# Features



**Slim but roomy**  
 The smart design gives Etac Clean a large and comfortable seating area even though the chair is neat and slim.




**Fits over the toilet**  
 Thanks to the upright operating handle, the chair is a perfect fit, even over wall-mounted toilets.



**Easy access**  
 The chair is open at the back to facilitate access for hygiene purposes.



**Foot support designed for feet**  
 The practical foot support has a soft, rounded shape which gives the feet extra comfort and protection.

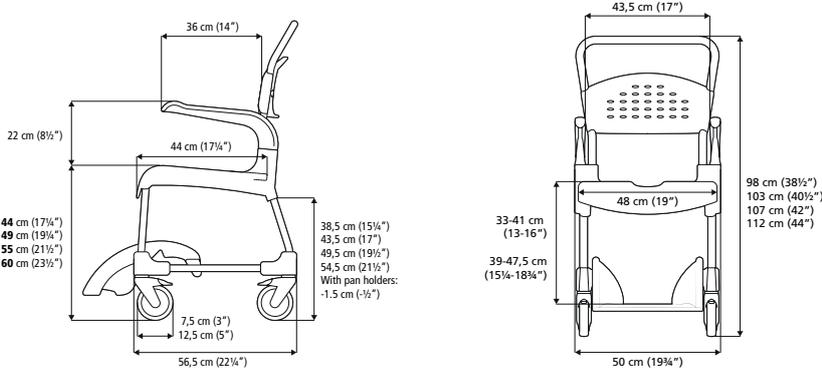



**Comfort and support**

The soft enveloping seat and back support provide stability, warmth and softness.

**Integrity in exposed situations**

Clean Comfort is designed to increase the feeling of integrity and dignity. The comfort padding protects and hides the user without reducing accessibility for the carers.


# Etac Clean Comfort

All the features of Etac Clean can also be found in this chair, but with even greater comfort. The seat, arm support and back support are padded and guarantee soft and comfortable seating.

## Soft and warmth

Etac Clean Comfort enhances the feeling of well-being for those who need a little extra comfort and support in the bathroom. The water-repellent and heat-reflective cover of the back support and arm supports helps the user to stay warm. The contoured seat offers comfortable support for the thighs and envelops the user for a safer and more comfortable feeling.

Etac Clean Comfort has the same basic benefits as other members of the Clean family – but now with even greater comfort.

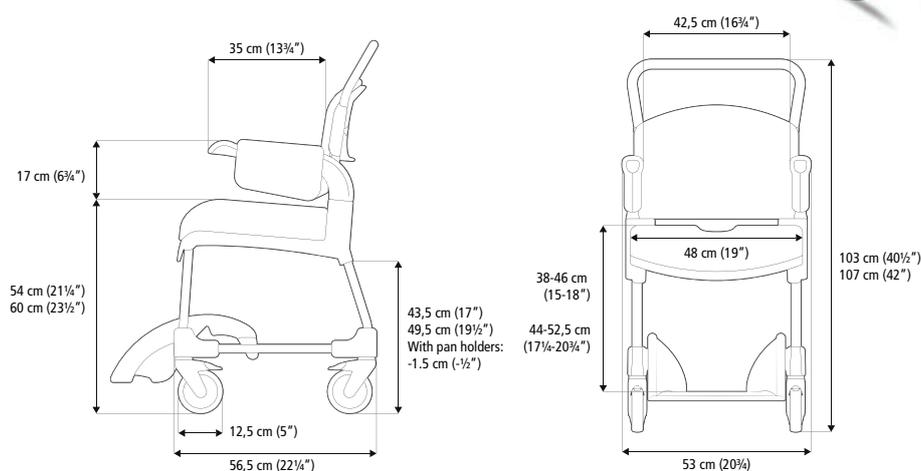

### Etac Clean Comfort

9 cm (19 1/4"), incl. Comfort Cover, Comfort Seat Soft, 4 lockable castors, white

### Item no.

80229298

### Material

Comfort Cover: Polyethylene  
 Comfort Seat Soft: Polyurethane  
 Chassis: Powder-painted steel tubing  
 Seat / back support: Polypropylene  
 Arm supports: Polypropylene  
 Foot support: Polypropylene  
 Castor brackets: Polyamide  
 Castors, lockable, ball-bearing: Polyamide/TPE

### Opening: depth x width

36.9 x 18.2 cm

### Weight

49 cm: 13.5 kg

55 cm: 14 kg

### Maximum user weight

130 kg

### Maintenance



Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.



**The same chair for all heights**

The smart design means that the chair is a perfect fit above all wall-mounted toilets, regardless of height.

**Additional support for the legs**

In the event of additional need for leg and foot support, various accessories are available, e.g. split foot supports and calf supports.


# Etac Clean Height Adjustable

In addition to all the features of the basic model, this variant allows you to adjust the height of the seat.

## Adjust the height for a perfect fit

The possibility of adjusting the seat height means that Etac Clean Height Adjustable can be tailor-made to suit the user's needs – for example to make transfers easier or to fit perfectly over the toilet. Adjustment is simple and no tools are required.

Etac Clean Height Adjustable has the same basic benefits as the other members of the Clean family – but is also height-adjustable up to 60 cm, and has a greater range of foot supports and calf supports.

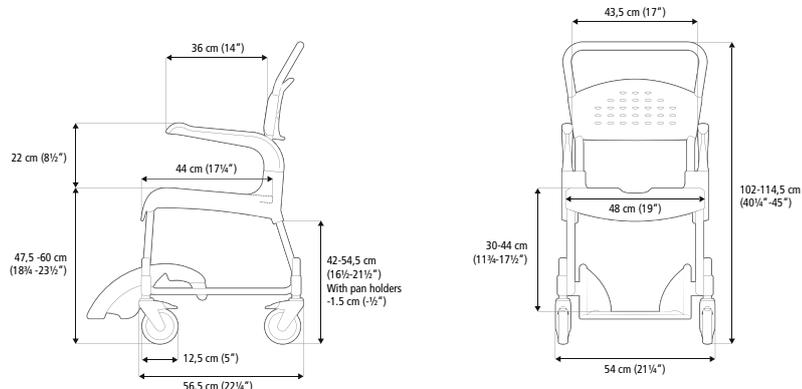

### Etac Clean Height Adjustable

4 lockable wheels, white

### Item no.

80229208

### Maintenance



### Weight

14.2 kg

### Maximum user weight

130 kg

### Material

Chassis: Powder coated steel tubing  
 Seat/Back support: Polypropylene  
 Arm supports: Polypropylene  
 Foot support: Polypropylene  
 Castors with ball, bearings: Polyamide/TPE/Stainless steel tubing  
 Castor height adjustment tubes: Stainless steel

Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution.  
 The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.



**Self propelled**

Easy to operate for the independent user.

**Safe and simple transfer**

The flat seat and the brake position facilitate lateral transfer.


# Etac Clean 24"

This variant of Etac Clean is suitable for those who can operate the chair independently and want to maximise their mobility.

## Easy to manoeuvre, for the independent user

The Etac Clean 24-inch wheels are positioned to make it easy to manoeuvre the chair without compromising user safety or comfort. It is easy to transfer to and from the chair thanks to the large, unrestricted seat in front of the wheels. The brake handles are also positioned below the seat so that they do not get in the way during transfer, while remaining easy to reach.

Etac Clean 24" has the same basic benefits as the other members of the Clean family – but allows the user to operate the chair independently.

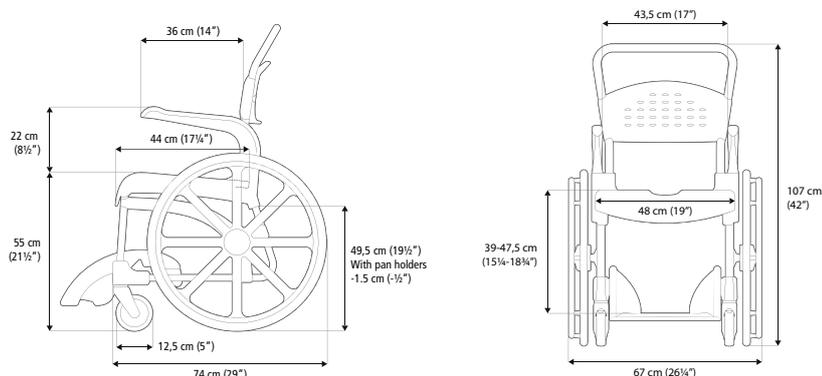

### Etac Clean 24"

- 55 cm (21¾) with 24" rear wheels, white
- 55 cm (21¾) with 24" rear wheels, lagoon green

### Item no.

- 80229276
- 80229216

### Material

- Rear wheel adapter: aluminum
- Rear wheels: polyamide, solid tires
- Chassis: Powder-painted steel tubing
- Seat/back support: Polypropylene
- Arm supports: Polypropylene
- Foot support: Polypropylene
- Castor brackets: Polyamide
- Castors, ball-bearing: Polyamide/TPE

### Weight

20 kg

### Maximum user weight

130 kg

### Maintenance



Clean / disinfect the product using a standard non-abrasive cleaning agent with a pH between 5 and 9, or with a 70% disinfectant solution. The product can be safely cleaned in a cabinet washer-disinfector at 85 °C for 3 minutes.

## Accessories



### Soft back support

0209225  
Adjustable.  
Washing at max 60° C (140° F).



### Soft pad

Grey: 80209260  
Heat-reflecting with a non-slip surface.  
Recess width: 17 cm (6¾"). Material: polyethylene. Cleaning max max. 60° C (140° F) or with alcohol wipe.



### Side Support

80209509  
Adjustable in width. Material: Polyester fabric, Velcro, polyethylene. Washing at max. 60° C (140° F). *Can not be used in combination with: 80209225.*



### Soft back pad

Grey: 80209261  
Heat-reflecting with a non-slip surface.  
Material: polyethylene. Cleaning max. 60° C (140° F) or with alcohol wipe.



### Crossbar

80209246  
Attached to the arm supports. Material: Aluminium, cellular polyethylene, polypropylene. *Can not be used in combination with: 80209228, 80209237 and 80209226.*



### Seat cover

80209247  
When a complete seat is required. Material: Polyurethane. Cleaning max. 85° C (185° F) or with alcohol wipe.




### Chest harness strap

0209442  
Adjustable in length. Attach to soft back support. Material: Polyester.




### Comfort Kit

80209237  
Includes both Comfort Seat Soft and Comfort Cover. Cleaning max. 60° C (140° F) or with alcohol wipe. *Can not be used in combination with: 80209225 and 80209246.*

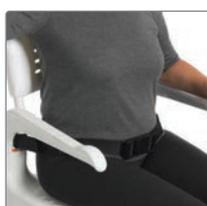

### Positioning belt.

80209440  
Adjustable in length. Attachment with velcro straps. Material: Polyester.

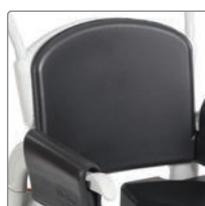

### Comfort Cover

80209228  
A soft back support with integrated arm support pads. Material: polyethylene. Cleaning max. 60° C (140° F) or with alcohol wipe. *Can not be used in combination with: 80209225 and 80209246.*



### Safety strap

80209034  
Attachments on the rear side of the back support. Always use together with positioning belt. Material: Polyester. *Can not be used in combination with: 80209225.*



### Comfort Arm Pads

80209226  
Soft and warm, leaving the handles free to ensure a firm grip when standing up. Material: polyethylene. Cleaning max. 60° C (140° F) or with alcohol wipe. *Can not be used in combination with: 80209246.*



### Cushion, protect

80209245  
Air-inflated ROHO cushion with excellent pressure distribution properties. Adjustable air pressure. Recess width: 21 cm (8¼"). Material: Neoprene. Cleaning max: 60° C (140° F).



### Comfort Seat Soft

Small opening width 15 cm (6"): . 80209301  
narrow opening width 18 cm (7"): 80209227  
A soft contoured seat. Material: polyurethane. Cleaning max. 85° C (185° F) or with alcohol wipe.

## Accessories



### Soap dish (does not fit Clean 24")

Grey: 80209266  
Attaches to the side of the seat. Built-in shower handle holder.  
Material: Polypropylene.  
(Does not fit Clean 24")



### Bucket holder

Grey: 80209258  
Fits Bucket with handle and lid and other buckets with a thick rim. Material: Polypropylene.



### Pan

Grey: 80209254  
Material: polypropylene.  
Pan incl. lid with handle  
Grey: 80209255  
Material: Polypropylene.



### Attachment for low mounting of pan/bucket

Grey: 80209259  
Fitted to the pan/bucket holder. Lowers the pan/bucket holder with 6 cm (2½").  
Material: Polypropylene.



### Pan holder

Grey: 80209257  
Fits both pan, bedpan and regular hospital pans with a thin rim.



### Disposable pans

80209282  
Strong and firm. 50 pcs.



### Bucket with handle and lid

80302026  
The bucket holder is not included.  
Material: Polypropylene.



### Disposable pan holder kit

Grey: 80209229  
Fits Disposable pan.

## Additional accessories – Clean Height Adjustable only



### Soap dish (does not fit Clean 24")

Grey: 80209266  
Attaches to the side of the seat. Built-in shower handle holder.  
Material: Polypropylene.  
(Does not fit Clean 24")

## Additional accessories – Clean 49 and 55 cm only



### Attachment for low mounting of pan/bucket

Grey: 80209259  
Fitted to the pan/bucket holder. Lowers the pan/bucket holder with 6 cm (2½").  
Material: Polypropylene.



### Pan

Grey: 80209254  
Material: polypropylene.  
Pan incl. lid with handle  
Grey: 80209255  
Material: Polypropylene.



### Disposable pans

80209282  
Strong and firm. 50 pcs.



### 75 mm (3") castors

White, front, pair: 80209515  
White, back pair: 80209516  
Lagoon green, front, pair: 80209513  
Lagoon green, back, pair: 80209514  
Reduce seat height by 5 cm (2").  
Mounted on Clean 44 cm as standard.  
Material: polyamide/TPE.



Etac is a world-leading developer and provider of ergonomic assistive devices and patient handling equipment. Our heart lies in the solutions that optimise quality of life for the individual, their family and caregivers.

For the latest news and continuously updated product information – visit: [www.etac.com](http://www.etac.com)

Etac AB  
Kista Science Tower  
SE-164 51 Kista, Sweden  
Tel +46 371 - 58 73 30  
[info@etac.se](mailto:info@etac.se) [www.etac.com](http://www.etac.com)

  
Creating Possibilities