

Push[®] care Knee Brace

Compression and support for arthritis

- Allows upward movement of patella while supporting both sides
- Can be adjusted for pressure
- Prevents brace slippage via internal silicone strips
- Prevents unwanted pressure on the patella tendon
- Allows easy step-in application due to open thigh design
- Can be machine washed on gentle cycle at low temperature
- Creates soft, comfortable feel and wicks moisture through the Sympress[™] microfibers



Indications

- Osteoarthritis/rheumatoid arthritis
- Patellofemoral pain or patella instability
- Mild functional instability

Measuring / Sizing

Bend your knee 90° and measure around the narrowest part of your knee.




SIZE	INCHES	CENTIMETERS
1	11 - 12 1/4	28 - 31
2	12 1/4 - 13 1/8	31 - 34
3	13 1/8 - 14 1/2	34 - 37
4	14 1/2 - 16 1/8	37 - 41
5	16 1/8 - 17 3/8	41 - 45




Easy application!

Pull the brace up over the knee until the recess is over the patella

Secure the brace by closing the hook & loop fasteners on each side of thigh (1) and (2)

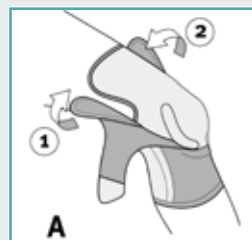

Close the straps diagonally across the upper part on the thigh, (3) then (4)

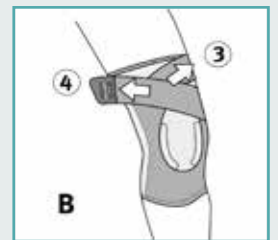

Insurance Information

- The innovative characteristics for some of the Push braces do not fit current L-code identifier categories; therefore, there is no L-code identifier for the Push care Knee Brace.
- BraceLab is not a provider. We do not file insurance nor can we provide information on insurance coverage or reimbursement rates. Please contact your insurance for filing information.



Available in the US from **y y y Wj gY tk j vUwHfEqo**
lphB vj gy tk j vUwHfEqo · P: 601-892-3115 · F: 601-892-3116

