

# Push<sup>®</sup> ortho Ankle Foot Orthosis (AFO)

Recreates natural gait

- Lifts foot in swing phase and controls foot stability in stance phase of walking
- Allows natural movement of toes and forefoot
- Supports both sides of the ankle, controlling inversion / eversion
- Secures foot to orthosis, allowing a wide variety of shoe styles
- Serves as night splint, if desired
- Can be machine washed on gentle cycle at low temperature
- Must be custom fitted by a trained professional



## Indications

- Drop foot with intact plantar flexion (ability to move foot downward)

## Contraindications

- Loss of plantar flexion
- Moderate to severe spasticity
- Extreme foot deformity



## Measuring / Sizing

Measure around the ankle as shown.

Please note that you should consult a trained professional for fitting and customization.

BraceLab cannot accept brace returns that have been worn or altered.



### Sizing: Right or Left Ankle

SIZE	INCHES	CENTIMETERS
1	10 1/2 - 12 1/4	27 - 31
2	12 1/4 - 13 1/8	31 - 34
3	13 1/8 - 15 3/4	34 - 40

## Insurance Information

- The assigned L-code for the Push *ortho* Ankle Foot Orthosis (AFO) is L1930.
- BraceLab is not a provider. We do not file insurance nor can we provide information on insurance coverage or reimbursement rates. Please contact your insurance provider for filing information.



Available in the US from [www.thewrightstuff.com](http://www.thewrightstuff.com)  
[info@thewrightstuff.com](mailto:info@thewrightstuff.com) · P: 601-892-3115 · F: 601-892-3116

