

# Nimbeze™

Simply nimble.

## User Instructions

### Shoe Helper

The nimbeze™ Shoe Helper is also a handy, ergonomic Reacher and can be used to reach and grab household items up to 16 ounces in weight.

When your Shoe Helper arrives it is in a contracted – or non-telescoped - position. To ready for use, simply pull the lower and upper halves in opposite directions to extend the tool. When fully extended it will lock into place. You will both feel and hear the locking mechanism engage. When in the correct and locked position for use, the trigger (on the handle end) will properly operate the grabbing claw on the lower end. It's that simple!



**NOTE:** For safety reasons the Shoe Helper is **not intended to be retracted into the shortened position in which it arrived**. It is possible to retract for storage by depressing the locking release button on the bottom of the center guard. However, the Shoe Helper will only operate in its telescoped and locked position.