



4. The swing-away armrests can be released to accommodate transfers. To unlock, push the red button and raise the armrest up. To lock back down in place, push down on the armrest until it locks into place.



**Note:** Ensure the chair is level by rotating the rubber feet to accommodate the shape of the bathtub or shower so they are flat to the surface and the tension buttons on the legs are at the same height.

## IMPORTANT SAFETY INFORMATION

**NOTE: Failure to follow the noted safety instructions can lead to severe injuries.**

- Never use this device with individuals who are severely mentally, cognitively, physically or vision compromised.
- Do not submerge the remote control in water. This will void your warranty.
- Always attach seatbelt and adjust for a comfortable and snug fit around the user's hips.
- Never leave a compromised user unattended.
- Always ensure the legs are properly adjusted so the chair is level.
- NEVER exceed the maximum weight capacity of 330 lbs.




**Step 10. Now the Power Carousel Transfer Bench Shower Chair is ready to be used.**



## Steps of Operation:

**Step 1.** Make sure the chair position is correct before using. The frame side with the motor should be placed on the outer side of bathtub and the frame with flat indentation should be placed inside the bathtub.

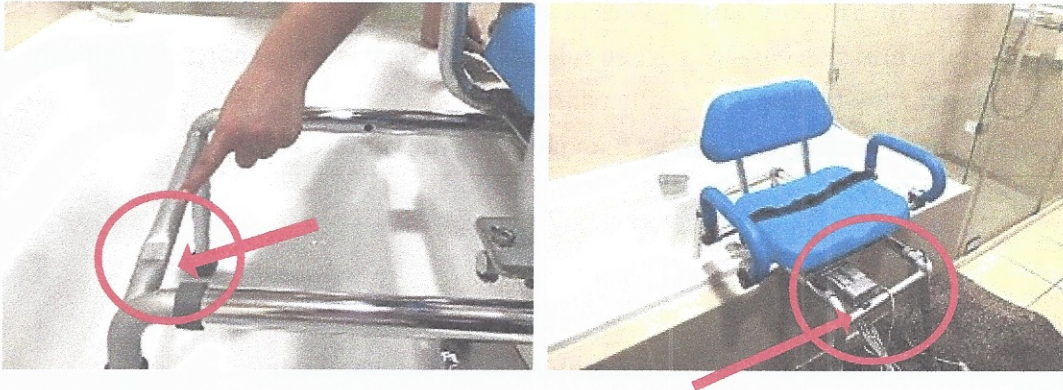


**Step 2.** Push red lever on the side of the seat and turn the seat facing the user. Once user is seated with seat belt attached, push down on the red lever and have the user rotate into the bathtub one foot at a time. **NOTE: Chair will lock into place every 90 degrees.**






# PlatinumHEALTH



**Step 5.** Cut off the white plastic tie and the black plastic tie around the motor before mounting legs.

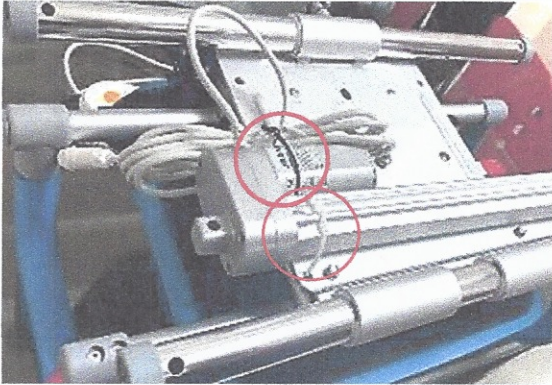

**Step 6.** Attach the four adjustable legs into each of the four side frame legs until the tension buttons lock in place. **Note:** Ensure the chair is level by rotating the rubber feet to accommodate the shape of the bathtub or shower floor.

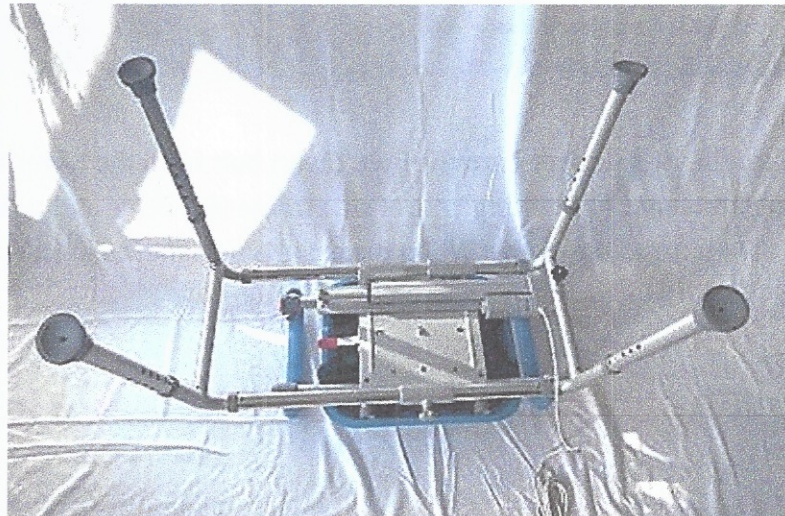



**Step 7.** Flip the chair to a standing position. Insert the backrest into the two receiving joints on the back side of seat and ensure the tension buttons lock in place.

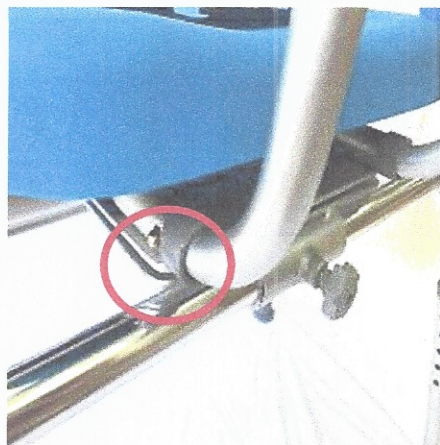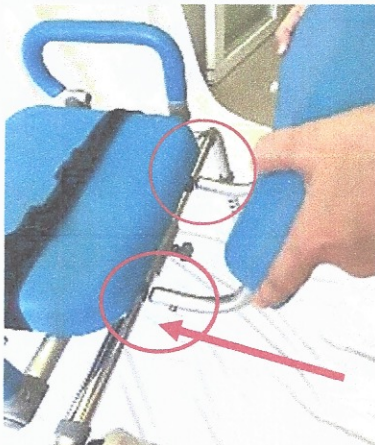



# PlatinumHEALTH

**Step 1.** Turn the seat (A) upside down. Position the side frame (D) to the side of the seat with the motor. Insert the side frame (D) into the two receiving joints on the seat frame. Be sure the tension buttons lock into place and can be seen protruding through the holes on the seat frame.

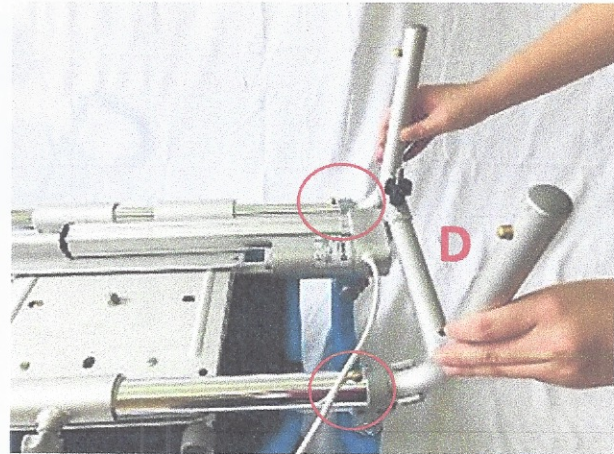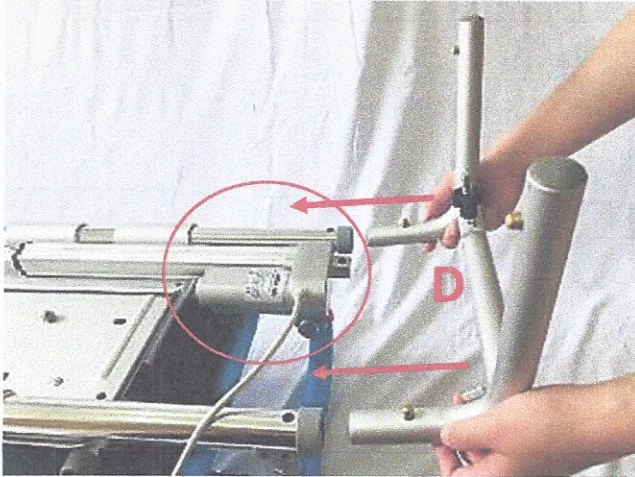

**Step 2.** Loosen and remove the black knob, bolt and ring washer attached on the side frame (D).

