

seat boost



Works best with:

- Arm chairs
- Sofas
- Benches
- Pews
- Theater seats
- Stable chairs with arm rests

Not recommended for:

- Cars
- Rocking chairs
- Recliners
- Stools
- Non-locking wheel office chairs

INSTRUCTIONS

Set Up

1. On a stable surface, hold the Seat Boost vertically so that the logo is against the surface and the handle is at the top
2. Push the cushion firmly - down and forward away from the base. The cushion will bend as you separate it from the base.
3. Find the weight setting closest to your weight. Keep in mind that a lower weight setting means less lifting assistance is provided. The setting chosen should be consistent with your needs.
4. Move the hydraulic lifting bar to the appropriate notch on the weight settings.
5. Check that the bar is securely in place before use to avoid injury.



Usage

1. The Seat Boost should be open and secured in the correct weight setting before use.
2. Place on a stable seating surface, with the logo facing front. The cushion may hang over slightly at the front of the seating surface.
3. Sit on the cushion near the top for a slow, gentle lowering.
4. To stand, lean forward, with hands on the arms of the chair. This will engage the lifting mechanism to help you rise smoothly and slowly to a standing position.
5. For travel and storage, follow steps 1-2 of the set up instructions, opening until the hydraulic lifting bar is released from the notch of the weight setting. Close for compact storage or easy carrying with the built-in handle.

Trouble Shooting

The Seat Boost may become stiff and not lower if not used regularly. To fix - set the weight at the lowest setting and sit on a few times until it moves freely, then adjust back to your appropriate weight setting.

CAUTION AND CARE

- Seating surface must be stable and non-moving
- Shifting weight forward will engage the lifting mechanism
- Additional cushions on top of the device are not recommended
- Brace yourself and be careful to rise slowly
- Discontinue use if the device is malfunctioning in any way
- Always set your accurate weight setting
- Removable cover is machine washable
- Hand wash cushion and air dry - Do not submerge
- Always use caution when using this device
- For ages 15 and up
- Warranty: 1 year from date of purchase