

Safe•t mate

Product use and specification sheet

Model Number	SM-008
Brand Name	Safe•t mate ®
Description	Adjustable Wheelchair Rear anti-tippers
Manufacturer	210 Innovations, LLC 210 Leonard Drive Groton, CT 06340 860.445.0210
Recommended Use	Safe•t mate Rear Anti-tippers are recommended for wheelchair users who exert excessive rearward force causing the chair to tilt while seated in, standing from or sitting into the wheelchair. Also suggested for use in conjunction with Safe•t mate Wheelchair Anti-rollback.
Not recommended for	Wheelchairs on which bottom horizontal frame tube is larger than 1" diameter.
Operation	Rear Anti-tippers are installed and adjusted to a position that causes them to make contact with the floor when the wheelchair is tilted backward. Thus preventing chair from tilting and tipping over backward.
Specifications	Mounting brackets are chrome plated 11 gage steel. Legs are heavy duty chrome plated rectangular steel bar stock. Wheels are 2" diameter X 1" wide plastic. Hardware is Grade 5. Mounting requirements / Adjustment Up to 1" round tubing. Mounts where horizontal and vertical rear frame members intersect. Minimum horizontal frame ground clearance is 2" Wheel height is adjustable between 2" and 12" from bottom of horizontal frame tube. Rear extension at lowest height is 15" back from mounting location.
Care	Check mounting and adjusting hardware for tightness periodically. No special cleaning instructions.