

Safe•t mate

Product use and specification sheet

| | |
|---------------------|---|
| Model Number | SM-011 |
| Brand Name | Safe•t mate ® |
| Description | Adjustable Wheelchair Front Anti-tippers |
| Manufacturer | 210 Innovations, LLC 210 Leonard Drive Groton, CT 06340 860.445.0210 |
| Recommended Use | Safe•t mate Front Anti-tippers are recommended for wheelchair users who exert excessive forward leaning force causing the chair to tilt forward while seated in, standing from or sitting into the wheelchair. |
| Not recommended for | Wheelchairs on which bottom horizontal frame tube is larger than 1" diameter. |
| Operation | Front anti-tippers are installed and adjusted to a position that causes them to make contact with the floor when the wheelchair is tilted slightly forward. This contact will discourage the wheelchair from tilting further and tipping over. This device may not prevent the wheelchair occupant from leaning forward enough to fall from the wheelchair. |
| Specifications | Mounting brackets are chrome plated 10 gage steel. Legs are heavy duty chrome plated tubular steel. Wheel height has 5 possible positions. Wheels are 2" diameter X 1" wide plastic. Hardware is Grade 5. Mounting requirements / Adjustment Up to 1" round tubing. Mounts on lower horizontal frame tubing directly behind front casters. Minimum horizontal frame ground clearance at mounting location is 8" and maximum is 12". |
| Care | Check mounting and adjusting hardware for tightness periodically. No special cleaning instructions. |