



# Gel Ovarations™

## Crutch Top Cover

**Gel and Foam**  
stretches over existing pad  
to add comfort to all  
standard crutches

[www.gelovations.com](http://www.gelovations.com)

- Improves comfort and decreases pressure
- Highest quality products on the market
- Easy to install – no tools required
- Washable neoprene/nylon non-slip cover
- Latex free

### ***Installation Instructions***

This product is designed to fit right over the existing foam top.

Open the Gel Ovarations™ Crutch Top Cover by undoing the Velcro® so that the 'soft' loop side of the Velcro® faces up. Place over the existing foam top cover. You may need to stretch the ends over the ends of the foam cover. Stretch the 'loop side' tab underneath the crutch top, then stretch the 'rougher hook side' tab underneath, stretch tight, and press it closed against the loop to connect the two parts of the Velcro®. One small section at a time disconnect and straighten the seam until it is 'flat and tight'.

### ***Cleaning***

Do NOT put pads/covers in a washing machine.

Do NOT submerge them in water or agitate.

Do NOT put in clothes dryer.

With HEATED DRYING turned OFF, you may wash them in the top rack of an automatic dishwasher.

Alternatively, wipe the surface with any normal household soap or cleaning solution that is not irritating and to which you are not allergic.

### ***Limited Warranty***

Gel Ovarations™ warrants that this product will be free of manufacturing defects for a period of 12 months. If defective, please return to place of purchase for an exchange or full refund. This warranty does not extend to abuse of the product such as tampering or modification.

Gel Ovarations™

[www.gelovations.com](http://www.gelovations.com)



Product Code: **CTC**