8-INCH  
WHEELS

Wonder  
ROLLATOR  
Plus 8" Wheels

Handle height adjusts 32.75" - 35"

La altura se ajusta de 32.75 - 35 pulgadas

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Supports up to 250 lbs.  
Soporta hasta 250 libras  
Supporte jusqu'à 250 livres

Locking hand brakes  
Frenos de mano con bloqueo  
Freins à main verrouillables

Large 8-inch wheels  
Grandes ruedas de 20,32 cm.  
Grandes roues de 20,32 cm

California law requires the following notice:

**WARNING:** Cancer and Reproductive Harm - [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

**ADVERTENCIA:** Cáncer y Daño Reproductivo - [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)  
**AVERTISSEMENT:** Cancer et effet nocif sur la reproduction - [www.P65Warnings.ca.gov](http://www.P65Warnings.ca.gov)

STW 1.0 081924

STANDER  
BE INDEPENDENT.

Wonder  
ROLLATOR  
Plus 8" Wheels

FOR EASIER MANEUVERABILITY  
IN ROUGH TERRAIN

PERFECT FOR

FOLDS FOR STORAGE  
SUPPORTS UP TO 250 LBS.

BE INDEPENDENT

SOYEZ AUTONOME

SE INDEPENDIENTE

Patent [www.stander.com/ip](http://www.stander.com/ip)

## WARNING: PLEASE READ BEFORE USING THIS PRODUCT.

Serious injury or death can result if the Rollator is not used in the intended manner. Stander, Inc. is committed to providing safe products and informing the user of ways to prevent accidents.

### SAFETY PRECAUTIONS:

- ✓ Regularly check the Rollator to make sure there are no broken or loose parts.
  - ✓ When traveling, make sure Rollator is not subject to being thrown or dropped. This could result in broken components which will make the product unsafe.
  - ✓ Individuals should not use this product if they are in a high risk condition. High risk users include those who are not mentally coherent (including those individuals under medication), users who exceed weight limit conditions, or any condition that impairs their ability to use the Rollator.
  - ✓ If you have any concerns or questions regarding these precautions, please contact Stander at 1-800-506-9901.
1. If you are suffering from an ailment or injury or are taking medications that affect your balance or ability to stand or sit down, you should first consult with a physician before using this product. If the product is not used properly, serious injury or harm could result.
  2. The product was not intended to carry the full weight of an individual. This product is only intended to provide balance and support while walking, sitting, and standing.
  3. Children should not be allowed to hang on or play with the product.
  4. **DISCLAIMER:** Neither Stander, Inc. nor the manufacture is liable for any harm or injury that may result or has resulted from the improper use or assembly of the product.
  5. Proper maintenance is important. Check the brakes during each use of the product to insure both braking and brake locking are functioning properly. The brake cable will stretch over time and brake pads will wear over time. This will diminish the braking capacity of the Hand Brake system. There must be sufficient engagement
  6. Weight Limit: 250lbs (113kg)
  7. This is a walking aid and not to be used as a transportation device.

### LIMITED LIFETIME WARRANTY

Stander, Inc. ("Stander") warrants to the original purchaser that the Product will be free from material defects in material or workmanship for the lifetime of the product. This limited lifetime warranty applies only when the Product is used consistent with the Product's specifications and is non-transferable and shall solely be applicable to the original purchaser.

In the event of a warranty claim, the original purchaser must notify Stander, during the warranty period, in writing of the alleged defect in workmanship or material and must ship the Product, along with the proof of purchase, to Stander. All shipping costs must be paid by the original purchaser. Stander will not refund or pay for any shipping charges.

If Stander determines, upon its sole determination, that there is a material defect in the material or workmanship of the Product, Stander will either send replacement parts or if needed replace the product and ship a replacement product or parts to the original purchaser.

The limited lifetime warranty does NOT apply to any Product:

- (1) That has been damaged in any way.
- (2) That has been altered or modified in any way or is used with any unauthorized attachments.
- (3) That was not properly installed or that was not used in accordance with the instructions provided by Stander.
- (3) That was used for purposes other than those for which the Product was designed or intended; or
- (4) Damaged due to abuse, misuse, mishandling, neglect, misapplication, accident or abnormal use or storage.

STANDER MAKES NO WARRANTIES OTHER THAN THOSE EXPRESSLY SET FORTH HEREIN. ALL OTHER WARRANTIES, EXPRESS OR IMPLIED INCLUDING ALL (I) IMPLIED WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, OR (II) IMPLIED WARRANTIES ARISING FROM ANY COURSE OF DEALING, USAGE OF TRADE, ADVERTISING OR PERFORMANCE OF PRODUCTS TO STANDARDS SPECIFIC TO THE COUNTRY OF IMPORT, ARE HEREBY EXPRESSLY DISCLAIMED.

THIS LIMITED LIFETIME WARRANTY IS THE FULL AND COMPLETE STATEMENT OF STANDER'S LIMITED LIFETIME WARRANTY OF THE PRODUCT, CONSTITUTES THE ENTIRE AND COMPLETELY INTEGRATED LIMITED LIFETIME WARRANTY OF STANDER, SUPERSEDES ALL OTHER STATEMENTS, AND CANNOT BE VARIED BY ANY ORAL OR OTHER WRITTEN STATEMENT.

THE EXCLUSIVE REMEDY FOR BREACH OF THIS LIMITED LIFETIME WARRANTY IS TO RETURN THE PRODUCT FOR THE REPLACEMENT OF THE NONCONFORMING PRODUCT.

UNDER NO CIRCUMSTANCES SHALL STANDER, INC. BE LIABLE FOR ANY SPECIAL, INCIDENTAL, EXEMPLARY, PUNITIVE, CONSEQUENTIAL DAMAGES, OR OTHER DAMAGES, BASED UPON BREACH OF THIS LIMITED WARRANTY, BREACH OF CONTRACT, OR STRICT LIABILITY.

Some states do not allow limitations on how long an implied warranty lasts. This warranty gives you specific legal rights, and you may also have other rights which vary from state to state. To obtain service under this Limited Warranty, you can reach Stander at (800) 506-9901 or (435) 755-0453. **DISCLAIMER:** Neither Stander nor the manufacturer will be liable for any harm or injury that results from the improper use or assembly of the product. Do not use or attempt to install this product unless you read and understand these warnings and follow all instructions.

Lifetime warranty does not include normal wear and tear items such as brake pads, hand grips, wheels, ski glides, etc. Replacements parts are available for purchase as needed.

**PLEASE READ SAFETY PRECAUTIONS AND INSTRUCTIONS BEFORE USING THIS PRODUCT.**

## OPENING THE ROLLATOR

**MAINTENANCE:** The rollator has lubricant applied on the plastic joints that slide along aluminum tubing. This lubricant will eventually dry out and make the rollator more difficult to open and close. When this occurs, a silicon spray between the plastic joints and the aluminum tube. This can be obtained from local hardware stores.

To open the rollator, undo buckle, pull handles apart, and push down on the seat as shown in Figure 1.

**WARNING:** Keep hands away from center pivot of the seat to avoid any potential pinch points.

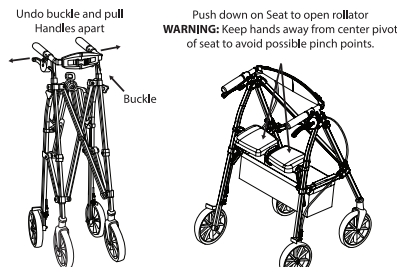
Figure 1

## CLOSING THE ROLLATOR

To close the rollator, pull up on the handle between the seat pads. Push handles together, and snap the buckle as shown in Figure 2.

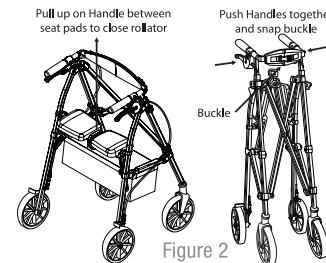
Figure 2

## HANDLE HEIGHT ADJUSTMENT:

With the rollator in the open position, remove knobs. Position the handle to the desired height, by aligning the holes in the handle tubes. Reinsert and tighten knob as shown in figure 3. Handle height adjusts 32.75"-35."

## BRAKE OPERATION:

Handle brakes have two functions as shown in Figure 3. Pull up on Brake Lever to provide braking action. Push down on Brake Lever to lock the brakes to prevent the rollator from moving while sitting. **WARNING:** Brakes must be in locked position before sitting. Do not move Rollator while seated.

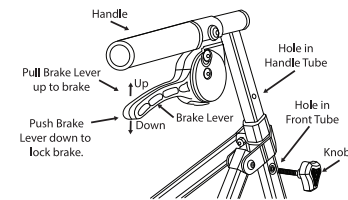
Figure 3

## ADJUSTING TENSION IN THE BRAKE CABLE:

When the Brake handle can no longer provide sufficient braking resistance, the Cable tension will need to be adjusted. Twist cable thumb screw in the counterclockwise direction to promote closer distance between the brake pad and the wheel.

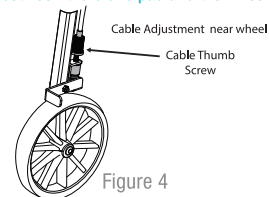
Figure 4

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