



Instructions



Snap It to the Walker, Drop the Cane Inside, & Go!

Designed with simplicity in mind, we made sure that attaching and using CaneTUBE® with any walker is as easy as pie.


CaneTUBE comes fully assembled. Just snap the two gripper clips onto the front support tubing of most walkers or rollators.

To Adjust:

If the fit isn't quite right, you can adjust CaneTUBE. The top spring clip can slide back and forth along the 12 inch slot.

To move the clip, loosen by turning the clip counter clockwise a couple of turns. Proceed to slide the clip to the desired position.

To tighten, turn the clip clockwise. A philips head screwdriver may be used to further tighten the screw, which is located at the base of the clip.

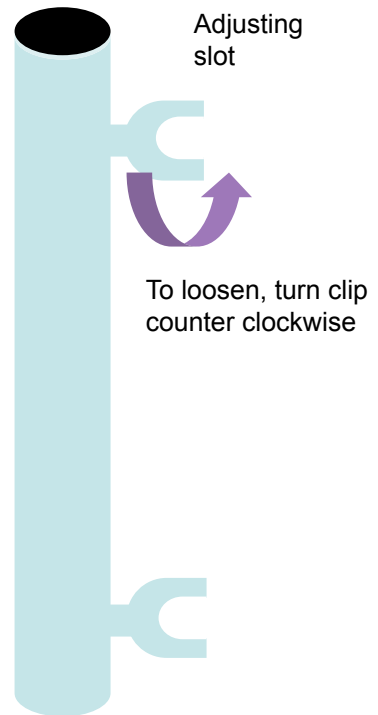

For more help or if you have questions, contact Customer Care at: **1-800-663-9037** or email us at **CustomerCare@CaneTUBE.com**