



## MOBILITY LIGHT

Model Number 00-100

### Installing/Changing Batteries

- 1) Unscrew black lens ring.
- 2) Lift off top cover of battery compartment.
- 3) Insert 2 AAA batteries as indicated.
- 4) Replace compartment cover, inserting back portion first.
- 5) Replace lens ring and follow operating instructions.

### Operating Instructions

Slide on/off switch to the on position.

The switch can remain in the on position, since the light and motion sensor now control the light. It is suggested that you turn the switch to the off position when storing the unit or packing it for travel.

**Customer Care Help Line – 1-866-235-5102**

[www.see4safety.com/helpline](http://www.see4safety.com/helpline)



### One-Year Limited Warranty

Your See 4 Safety™ Mobility Light is guaranteed to be free of manufacturing defects for a period of one year from date of purchase. If your mobility light fails to function correctly, return the unit postage prepaid, along with \$5.00 for return shipping and handling, to NEPA Partners LLC, 1400 Sans Souci Parkway, Hanover Township, PA 18706. Please include dated proof of purchase, a description of the problem and your shipping information. At our option, NEPA Partners LLC will repair or replace any unit found to be defective in materials or workmanship under normal use. **THIS WARRANTY DOES NOT COVER BATTERIES.**

### WARNING

- This product is not a toy and should not be stored or left in locations accessible to young children.
- Choking Hazard: This product contains small parts when disassembled.
- When not using this product for extended periods, remove batteries to prevent product damage due to battery leakage.
- Observe proper polarity when changing batteries. Lamp may not light if batteries are inserted incorrectly.
- To prevent danger from explosion or leakage, do not throw batteries into fire or expose them to excessive heat.